


# Q-Series 42" Expander Kit

(Kit# Q6709 for Rexx and Advantage beds)

The Span MC Expanders Kit part no Q6709 is supplied with form-fitted packaging for protective storage and reuse. The kit contains 6 expander units and two mattress retainers, one each for the center of the head and foot decks for additional stability with wider mattresses in some applications.

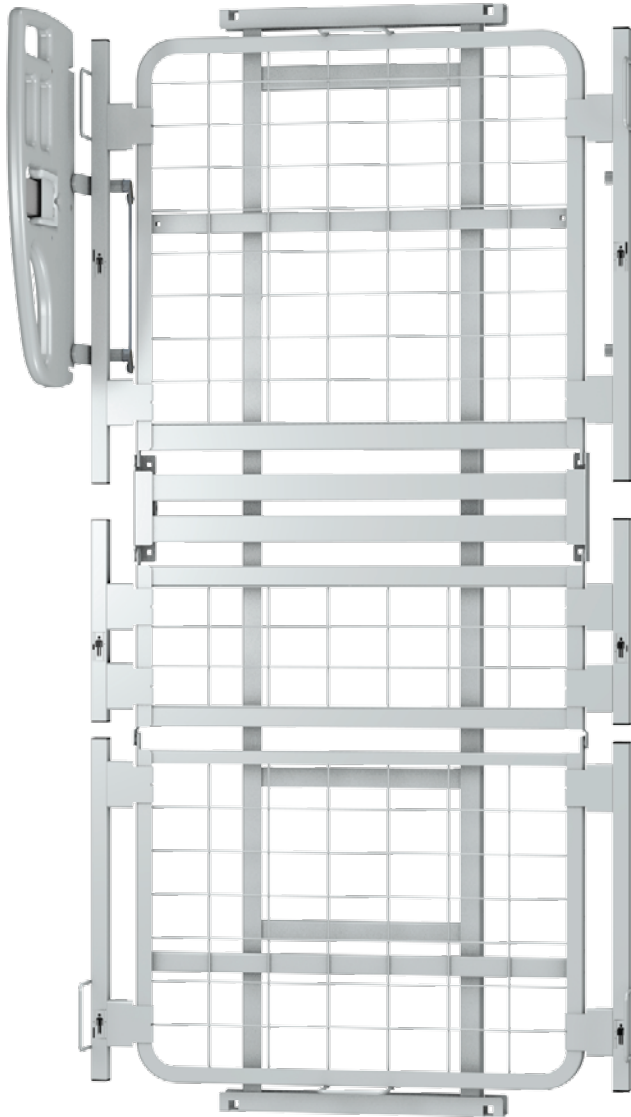

Bed frame, overhead view.

## STEP 1

If the bed is fitted with half head assist rails it is necessary to remove them and the four corner mattress retainers prior to installation of the expanders. The six expander units will then install toollessly, converting the sleep surface to a 42" width. We recommend storing the original corner mattress retainers in the kit for reuse.



## STEP 2

Note the placement decals on each of the six unique expander units and prepare to install them on either side of the head, seat and foot decks.



## STEP 3


Using the figure (upper left) as a guide, connect each expander unit to the upper surface of the grid deck and then lower the expander until the spring loaded retaining latch attaches securely to the lower portion of the deck.



**SPAN**  
better science. better care.



*The expanders are fully compatible with Span MC half head assist rails. The rails should be fitted to the head deck expander units prior to installation. See p. 2 for instructions.*

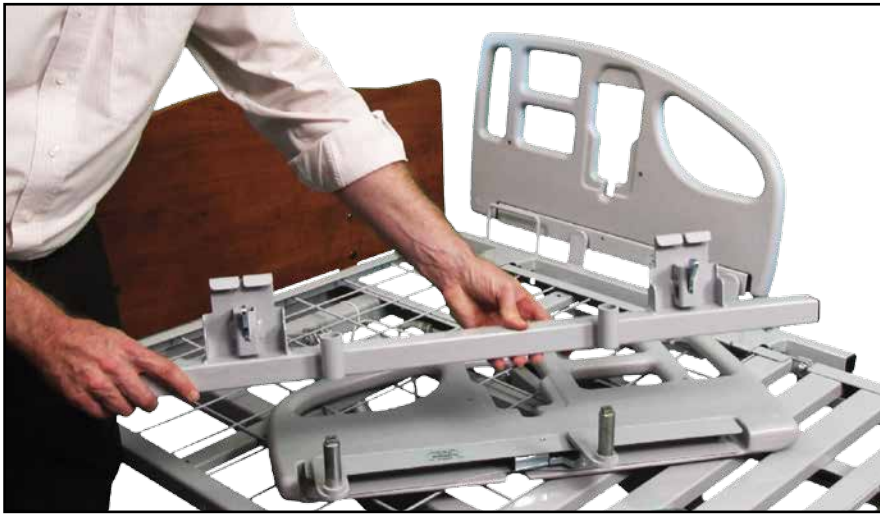

**To remove and install half head assist rails, follow these instructions which apply to molded plastic or steel rails:**

**Removal:** Using the 5/32" hex (Allen) wrench, remove the 1/4-20 X 5/8 flat head screws from the 1/2 rail assemblies, then remove the equalizer bar assembly, wave washers, and the mounting channel. The mounting channels are not used with this kit.



**Installation:** Install the right hand 1/2 rail into the mounting tubes on the underside of the right hand head deck expander, then re-install the wave washers, right hand equalizer bar assembly (each casting is marked "R") and 1/4-20 X 5/8 flat head screws. The equalizer bar assembly must be oriented to the raising arms. Verify that the rail swings through its full range of motion and that it will latch in the upright position. Repeat for the left hand side 1/2 rail.

## 42" Expander for REXX/Advantage Grid Deck - Kit Q6709

ITEM NO.	QTY.	PART NO.	DESCRIPTION
1	1	SK0002	EXPANDER, SERVICE ASSEMBLY, 42", HEAD DECK, LH
2	1	SK0003	EXPANDER, SERVICE ASSEMBLY, 42", HEAD DECK, RH
3	1	SK0004	EXPANDER, SERVICE ASSEMBLY, 42", FOOT DECK, LH
4	1	SK0005	EXPANDER, SERVICE ASSEMBLY, 42", FOOT DECK, RH
5	1	SK0006	EXPANDER, SERVICE ASSEMBLY, 42", KNEE DECK, LH
6	1	SK0007	EXPANDER, SERVICE ASSEMBLY, 42", KNEE DECK, RH
7	2	QS-3647	MATTRESS STOP, END
8	4	QP-2501	SCREW, MACHINE, HEX, #10-32 X 5/8
9	4	QP-2502	LOCKNUT, # 10-32 HEX

Contact your local Span Representative or  
call Beds/Furniture Customer Service:

800-268-8671

[www.spanamerica.com](http://www.spanamerica.com)

**SPAN**  
better science. better care.

