


# RISK MANAGER<sup>TM</sup> GRIP STRIPS

## OPTIONAL ACCESSORY

For use on highly-polished floors where extra slide resistance may be desired.

## INSTRUCTIONS FOR USE

1. **Clean** the underside of the Risk Manager<sup>TM</sup> mat with ammonia-based window cleaner to remove dust and dirt and improve slide resistance. Allow surface to dry before attempting to attach strips.
2. **Attach** blue strips to underside of Risk Manager mat as shown in the diagram below, as follows:

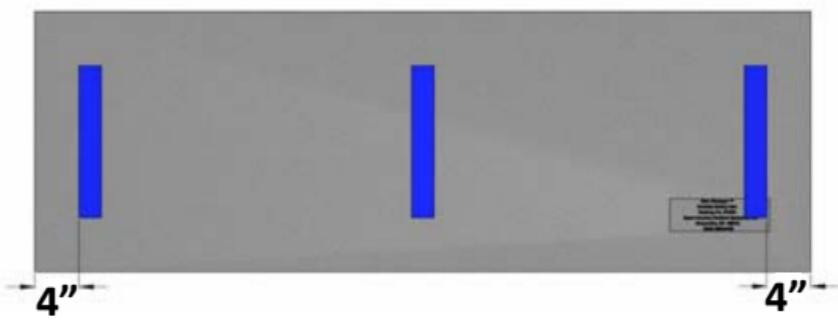

- Peel off the white backing paper from the first strip, exposing the adhesive. Take care not to allow strip to fold back upon itself.
  - Place strip into position at the center of mat. Press firmly to affix.
  - Peel off white backing paper from the second strip, and press into position approximately 4 inches from one edge as shown in the diagram. Repeat for the third strip.
3. **Expose** slide-resistant surface of strip by peeling off the clear film from the face of the blue side, starting at the pre-peeled corner. If necessary, use a small piece of clear adhesive tape to pick up a corner edge so that the clear film is easy to peel back.
  4. **Place** mat into position on clean, dry floor.
  5. **Step** on the pad to ensure adherence on floor. Evaluate slide resistance.
    - **Cleaning:** To maintain maximum slide resistance, wipe strips clean with water or window cleaner using a lint-free cloth to remove dust and contaminants. Allow to air dry before returning to floor.

RE-ORDER NO.: 43398  
DESCRIPTION: Grip Strips 2" W x 14" L  
QUANTITY: 3 per package

**800-888-6752**

[www.spanamerica.com](http://www.spanamerica.com)

**SPAN**

P09591 r1